# Aquasil® Ultra+

## Smart Wetting® Impression Material

Aquasil Ultra+ XLV • Aquasil Ultra+ LV • Aquasil Ultra+ Medium • Aquasil Ultra+ Heavy  
• Aquasil Ultra+ Rigid

### DIRECTIONS FOR USE – ENGLISH

For dental use only. U.S.A.: Rx only.

#### 1. PRODUCT DESCRIPTION

Aquasil Ultra+ Smart Wetting Impression Material is a quadrafunctional hydrophilic addition reaction silicone elastomeric dental impression material with excellent hydrophilic properties, dimensional accuracy, high tear strength, and resistance to permanent deformation.

#### 1.1 INDICATIONS FOR USE

Aquasil® Ultra+ Material is indicated for all dental impression techniques.

#### 1.2 Contraindications

Aquasil Ultra+ Material is contraindicated for use with patients who have a known hypersensitivity to peppermint oil or any of the components

#### 1.3 Delivery Forms (Some delivery forms may not be available in all countries)

Aquasil Ultra+ Material is available in:

- Light-bodied consistency, Medium-bodied consistency, and Heavy-bodied consistency
- digit® Targeted Delivery System Unit Dose Dispensed delivery, digit Power® Dispensed delivery, 50mL cartridge delivery, and 5:1 DECA™ delivery for Dynamic Mixing Machines.
- Fast Set, Regular Set and Extended Set

#### 1.4 Composition

Polydimethylsiloxane polymer; Polymethylhydrogen Siloxane; Silicon Dioxide; Sodium Aluminosilicate; Organic Platinum Complex; Surfactant; Titanium Dioxide; Fluorescent Pigments; Metallic Oxide Pigments; Iron Oxide Pigments; Organic Pigments; Peppermint Oil

#### 1.5 Technical specifications

- Classification: ISO 4823
- Type 3: Light-bodied consistency: Aquasil Ultra+ XLV, Aquasil Ultra+ LV
- Type 2: Medium-bodied consistency: Aquasil Ultra+ Medium, Aquasil Ultra+ Heavy
- Type 1: Heavy-bodied consistency: Aquasil Ultra+ Rigid
- Working Time (22°C/72°F) ..... **Fast Set:** 1'00" minimum  
**Regular Set:** 2'15" minimum  
**Extended Set:** 2'15" minimum  
(room temperature mixing and preparation, maximum insertion time)
- Working Time (37°C/98°F) ..... **Fast Set:** 35" Maximum  
**Regular Set:** 1'10" Maximum  
**Extended Set:** 1'45" Maximum  
(intraoral syringing prior to tray insertion)
- Setting Time (Mouth Removal Time) **Fast Set:** 2'30" from start of mix  
**Regular Set:** 5'00" from start of mix  
**Extended Set:** 5'30" from start of mix
- % Linear Dimensional Change ..... < 0.50
- % Recovery from deformation ..... ≥ 98
- Detail Reproduction ..... 20 microns

#### 1.6 Compatible tray adhesive

Aquasil Ultra+ Material is compatible with conventional tray adhesives designed for use with vinyl polysiloxane materials (See complete Directions for Use).

### 2. GENERAL SAFETY NOTES

Be aware of the following general safety notes and the special safety notes in other chapters of these directions.

#### Safety alert symbol

This is the safety alert symbol. It is used to alert you to potential personal injury hazards. Obey all safety messages that follow this symbol to avoid possible injury.

#### 2.1 Warnings

- Aquasil Ultra+ Material contains polymerizable monomers and surfactants which may be irritating to skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.
- Avoid Eye contact to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
- Avoid Skin contact to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs immediately remove material with cotton and wash thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.
- Avoid ingestion/swallowing of material to prevent irritation or obstruction. If accidental swallowing occurs, drink lots of water. This material is not hazardous when small quantities are ingested. Larger quantities may cause bowel obstruction. Seek medical attention in the event of digestive irregularities.
- Do not use Aquasil Ultra+ Material as a temporary reliner. Use of the material in this manner may cause irritation to the oral mucosa. If symptoms occur discontinue use immediately. If irritation does not subside consult a physician.
- Avoid multiple impressions at one sitting (more than three of the same arch/area) to prevent irritation to the oral mucosa. If symptoms occur discontinue use immediately. If irritation does not subside consult a physician.

#### 2.2 Precautions

- This product is intended to be used only as specifically outlined in the Directions For Use. Any use of this product inconsistent with the Directions For Use is at the discretion and sole responsibility of the practitioner.
- Insufficient data exist to support the use of Aquasil Ultra+ Material for impressions on exposed bone. Aquasil Ultra+ Material should only be used on or around intact mucosa.
- Wear suitable protective eyewear, mask, clothing and gloves. Protective eyewear is recommended for patients.
- Devices marked "single use" on the labeling are intended for single use only. Discard after use. Do not reuse in other patients in order to prevent cross-contamination.
- All products should be used at room temperature. Higher temperatures reduce work times and laboratory bench set times (faster), lower temperatures increase them (slower).
- To obtain optimum physical properties, the mix should be completely homogeneous (streak free).
- Aquasil Ultra+ Material should be used with properly fitted, rigid trays for most applications. Trays with retentive features such as Caulk Rim-Lock Trays or perforations, pretreated with tray adhesive, are recommended. To reduce gagging, choking or swallowing, do not overfill tray.
- Material should extrude easily: DO NOT USE EXCESSIVE FORCE. Excessive pressure may result in unanticipated extrusion of the material or cause cartridge rupture.

- Safety and efficacy of Aquasil Ultra+ Heavy, Aquasil Ultra+ Rigid Material when used as an intraorally syringed (wash) material has not been demonstrated.
- Due to the stiffness of Aquasil Ultra+ Rigid Material, severe undercuts should be blocked out prior to tray insertion. For full arch dentate impressions, select other Aquasil Ultra+ Material viscosities for tray material.
- Suitable cartridges for use with digit Power® Dispenser will be marked PWR on tab. Do not use digit Power® Dispenser with materials in digit® Targeted Delivery System cartridges designed for manual dispensing. Device and cartridge malfunction may occur.
- **Interactions:**
  - Do not use Hydrogen peroxide-containing dental materials in conjunction with this product since they may interfere with the hardening of the polymeric components.
  - Use of latex or "sulfur-containing" polymeric gloves, some hand soaps and lotions can lead to interference with setting reaction. Contamination may accumulate on the surface of the skin and may not easily wash off. Non-latex gloves worn during use will avoid this possibility.
  - Handle cords with college pliers. Do not touch prepared tooth with gloves. Rinse preparation with copious amounts of water before making the impression.
  - Aquasil Ultra+ Material should not be intermixed with, or used in conjunction with, other manufacturers' vinyl polysiloxanes or any polyether, polysulfide or conventional (condensation cured) silicones.
  - The dual phase, two-step ("putty/wash") crown and bridge impression technique is only recommended with Aquasil Putty (all versions) or Reprosil® impression materials (available separately) for primary tray impressions.

#### 2.3 Adverse Reactions

- Eye contact: irritation and possible corneal damage.
- Skin contact: irritation or possible allergic response. Reddish rashes may be seen on the skin.
- Bowel obstruction or other digestive distress may result from ingestion of mixed impression material (see Warnings).

#### 2.4 Storage conditions

- Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.
- Store at temperatures between 10°C/50°F - 24°C/75°F.
- Use the product at room temperature.
- Do not store used cartridges with new (unused) mix tips attached.
- Do not freeze.
- Protect from moisture.
- Do not use after expiration date.

### 3. STEP-BY-STEP INSTRUCTIONS

#### 3.1 Cartridge Dispensing Gun

- Insert cartridge into dispensing gun.
- Bleed cartridge before installing new mixing tip.
- Preparation of syringe/wash material:
  - Dispense directly to the mouth with the intraoral tip; OR
  - An intraoral syringe may be filled with mixed material from the cartridge; OR
  - Prepare digit Power® Dispenser or digit® Targeted Delivery System Unit Dose wash/syringe material according to the Directions for Use.
- **NOTE:** Do not inject Aquasil Ultra+ Heavy or Aquasil Ultra+ Rigid Impression Material intraorally.
- Disinfect cartridge dispensing gun, cartridge and attached used mix tip for storage.
  - **Immediately prior to subsequent usage,** remove the used mixing tip, bleed cartridge, and install a new mixing tip.
  - Dispose of cartridge when empty. Do not attempt to refill or consolidate used cartridges.

#### 3.2 digit® Targeted Delivery System Unit Dose

- Insert plunger into open end of syringe barrel.
- Snap circular disk off of end of cartridge.
- Insert unit dose cartridge into mix tip. Ensure both cartridge spurs fully lock into mix tip slots. Attach intraoral tip.
- Insert assembled cartridge/mix tip (with intraoral tip) into the syringe side opening. Slide mix tip through barrel end with cross notches. Pull on mix tip to ensure complete seating.
- Dispense the material by applying slow, steady pressure on plunger. Do not use excessive force.
- To disassemble, withdraw the plunger, tap mix tip against the counter to dislodge, and remove through side opening.
- Properly discard used cartridge/mix tip and prepare syringe for reuse.

#### 3.3 digit Power® Dispenser (see complete Directions for Use)

- Power handpiece on.
- Select pre-set timer.
- Assemble digit Power® cartridge. Ensure selected cartridge is marked PWR on tab.
- Assemble cartridge cap.
- Attach cartridge cap assembly to handpiece. Ensure cap is attached securely.
- Place a protective barrier sleeve over Dispenser.
- Bleed cartridge, then dispense material.
- Properly discard used cartridge/mix tip and reprocess according to the Directions for Use.

#### 3.4 DECA™ cartridge loading

**NOTE: Heavy Cartridge -** Dropping may result in damage to the cartridge rendering it unusable. The DECA™ cartridge fits duomix®, Pentamix, MixStar, Dynamix and Volume Mix dynamic mixers. Carefully remove dispensing cartridge from the package. Also refer to separate dynamic mixer manufacturer's instructions.

- Remove the cap from the cartridge.
- Insert the cartridge into the dynamic mixer.
- Bleed the cartridge before installing a new mix tip.
- Install a mixing tip. Use only Dentsply Sirona mix tips. Other tips may not seal properly.
- Place locking ring over the mix tip and turn until ring is locked in place.
- Dispense the mixed base and catalyst into a suitable tray.
  - **Immediately prior to subsequent usage,** remove the used mixing tip, bleed cartridge, and install a new mixing tip.

#### 3.5 Impression technique

Single or Dual Phase one-step ("double-mix") technique with a rigid tray or with double arch technique

**NOTE:** Tray and wash material mix timing – It is highly recommended to utilize an assistant to assure simultaneous syringing and tray loading.

- Assure hemostasis and clean the field with air water spray.
- **Syringe material:** Backfill the impression syringe or attach an intraoral tip to the end of the mix tip, or prepare digit® cartridge syringe/Dispenser.
- **Tray material:** Dispense material (assistant) into impression tray. Do not overfill tray.
- Simultaneous with tray loading, syringe (dentist) completely around the preparation/abutment.
- **NOTE:** Tray and wash material insertion timing
  - **Fast Set wash and tray material:** Loaded tray must be inserted within 35 seconds from time of first syringing of wash material intraorally.
  - **Regular Set wash and tray material:** Loaded tray must be inserted within 1 minute 10 seconds from time of first syringing of wash material intraorally.
  - **Extended Set wash and tray material:** Loaded tray must be inserted within 1 minute 45 seconds from time of first syringing of wash material intraorally.
- Insert filled tray into the mouth.
- Minimum Removal Time (from the start of mix) – **Fast Set:** 2 minutes 30 seconds; **Regular Set:** 5 minutes;

**Extended Set:** 5 minutes 30 seconds

- **NOTE:** If Regular Set Wash is used with Fast Set Tray material, Minimum Removal Time is 4 minutes from start of mix.
- Follow disinfection procedures before shipping and/or pouring.

<b>TIMING GUIDE</b>	<b>Fast Set:</b> up to 0:35 (35 sec)	<b>Fast Set:</b> 2:30 (2 min 30 sec)
0:00 (0 sec)	<b>Regular Set:</b> up to 1:10 (1 min 10 sec)	<b>Regular Set:</b> 5:00 (5 min)
	<b>Extended Set:</b> up to 1:45 (1 min 45 sec)	<b>Extended Set:</b> 5:30 (5 min 30 sec)

Begin simultaneous syringing and tray loading	Complete syringing and seat loaded tray	Remove set impression
---	---	-----------------------

### 3.6 Dual Phase two-step (“putty-wash”) technique with a primary tray impression

- Complete primary impression with Aquasil Putty or Reprosil® Tray material.
  - Place one or more polyethylene plastic sheets over the impression material surface. This sheet creates the uniform spacing required for this technique. It also protects the surface from contamination.
  - Insert filled tray into the mouth. Move tray slightly from side to side and back and forth. This creates the spacing for the wash material.
  - Remove tray from the mouth when material has reached a rubbery consistency and set aside. Do not disturb impression until fully set.
  - Remove polyethylene sheet, rinse impression with water and air dry.
- It is recommended to utilize an assistant and two cartridge guns, or one cartridge gun and digit® Targeted Delivery System Unit Dose syringe or digit Power® Dispenser to assure simultaneous syringing and tray loading.
- Follow impression steps above, using wash material for both intraoral syringing and loading into primary tray impression.

### 3.7 Impression technique: Edentulous/Overdenture impression technique

*Aquasil Ultra+ Medium, Aquasil Ultra+ Heavy, and Aquasil Ultra+ Rigid are acceptable for border molding.*

- **Border Molding Technique**
  - Dispense material on to the borders of the tray.
  - Insert the tray and perform conventional border molding techniques.
  - When material has reached a firm consistency, remove tray and examine borders. The addition of a small mix can be used if voids or deficient areas are present. Over-extensions can be easily removed with a scalpel blade.
- The final impression can be made with Aquasil Ultra+ XLV, LV, Medium, or Heavy material.
- Insert filled tray into the mouth.
- Observe the Minimum Removal Time of the selected Aquasil Ultra+ final impression material.
- Follow disinfection procedures before shipping and/or pouring.

### 3.8 Impression Technique: duplication of models

- Load the impression tray with Aquasil Ultra+ tray material.
- It is suggested to simultaneously syringe, using Aquasil Ultra+ wash material, completely around the model teeth.
- Seat model into the loaded tray. Retain impression in position until firmly set.
- Remove model from impression by pulling slowly to break seal.

## 4. HYGIENE

### Cross-contamination

- Do not reuse single use products. Dispose of in accordance with local regulations.
- Do not attempt to refill or consolidate used cartridges. Dispose of empty cartridges in accordance with local regulations.
- Reprocess reusable products as described below.

### 4.1 digit Power® Dispenser

For reprocessing instructions please refer to the Instructions for Use of the digit Power® Dispenser, which is available on our webpage at [www.dentsplysirona.com](http://www.dentsplysirona.com) and [www.dentsply.eu/IFU](http://www.dentsply.eu/IFU). If requested, we will send you a free printed copy of the Instructions for Use in the language you require within 7 days. In the USA or Canada, call 1-800-532-2855. All other countries, use the order form provided on [www.dentsply.eu/IFU](http://www.dentsply.eu/IFU) for this purpose.

#### For Manual Cleaning

- Use a new clean pair of examination gloves.
- Discard gloves according to local regulations and disinfect hands with an appropriate bactericidal, virucidal, and fungicidal hand disinfectant solution according to local regulations. Use according to disinfectant solution manufacturer’s Instructions for Use.
- Refer to Dynamic Mixer manufacturer’s instructions for dispenser cleaning and disinfection instructions.

### Instructions for disinfecting/sterilizing Dispenser Gun and Cartridge

<b>Warnings</b>	<ul style="list-style-type: none"> <li>• High level disinfection is appropriate for the Dispenser Gun and cartridge.</li> <li>• It is not recommended to submerge Dispenser Gun or cartridge into disinfection solutions. Discoloration and deterioration may occur.</li> <li>• Do not autoclave cartridges.</li> <li>• Vigorous wiping of cartridge can destroy the label. Wipe cartridge gently.</li> </ul>
<b>Limitations on Reprocessing</b>	<ul style="list-style-type: none"> <li>• Repeated processing has minimal effect on the Dispenser Gun.</li> <li>• The Dispenser Gun can be subjected to the manual cleaning and disinfection process up to 1000 times.</li> <li>• The Dispenser Gun can be subjected to the automated washer/disinfectant process up to 1500 times.</li> <li>• The Dispenser Gun can be subjected to the steam autoclave sterilization process up to 1000 times.</li> <li>• Cold liquid disinfection/sterilization, chemical vapor sterilization, and dry heat sterilization methods have not been tested or validated for efficacy and are not recommended for use.</li> </ul>
<b>Point of Use</b>	<ul style="list-style-type: none"> <li>• Remove excess soil with disposable cloth/paper wipe.</li> <li>• Prior to Cleaning and Disinfection, the Dispenser Gun should be disassembled:               <ol style="list-style-type: none"> <li>1. Place your thumb on the rear slide lock.</li> <li>2. Push upwards and release lock. Pull slide back.</li> <li>3. Lift latch securing cartridge, Remove cartridge.</li> <li>4. Remove slide plunger by withdrawing fully out the front of dispenser.</li> <li>5. Remove excess material with a soft paper tissue and alcohol of 70%.</li> </ol> </li> <li>• It is recommended that the dispenser gun and cartridge be reprocessed as soon as is reasonably practical following use.</li> </ul>
<b>Containment and Transportation</b>	<ul style="list-style-type: none"> <li>• No particular requirements.</li> </ul>

<b>Cleaning and Disinfection: Automated</b>	<ul style="list-style-type: none"> <li>• Use only a properly maintained, inspected, calibrated, and approved washer/disinfectant (according to ISO 15883).</li> <li>• Perform a cleaning program cycle (the following cycle is recommended):               <ul style="list-style-type: none"> <li>• 1 minute pre-wash/rinse in cold tap water.</li> <li>• 5 minutes wash with enzymatic detergent at 55°C.</li> <li>• 1 minute enzymatic neutralization in cold tap water.</li> <li>• 1 minute rinse in cold tap water.</li> <li>• Thermal disinfection with minimum AO of 3000 (e.g. 5 minutes at 90°C).</li> <li>• 20 minutes drying at 100°C.</li> </ul> </li> <li>• Follow manufacturer’s recommendation for use of detergent and neutralizer, observing concentrations and contact times.</li> </ul>
<b>Cleaning and Disinfection: Manual</b>	<ul style="list-style-type: none"> <li>• As an alternative to automated cleaning and disinfection, the dispenser gun has to be cleaned by scrubbing with hot water and soap or detergent. The cartridge must be cleaned with a single-use cloth soaked in hot water and detergent.</li> <li>• Remove cartridge and plunger from Dispenser Gun.</li> <li>• After cleaning thoroughly wipe all device surfaces with a single-use cloth in combination with a water-based, bactericidal, virucidal and fungicidal disinfection solution approved according to local regulations and use according to disinfectant solution manufacturer’s Instructions for Use. Make sure disinfectant solution is compatible with cleaning solution. Pay special attention to device seams and insertions.</li> <li>• Use of phenolic-, iodophor- or organic solvent (e.g. alcohol) based or containing products may over time cause surface staining.</li> <li>• Remove disinfectant solution residue with a cloth soaked with water.</li> <li>• Dry device with a lint-free single-use cloth.</li> </ul>
<b>Packaging</b>	<ul style="list-style-type: none"> <li>• Paper/plastic steam sterilization pouches may be used, but are not required.</li> </ul>
<b>Sterilization</b>	<ul style="list-style-type: none"> <li>• Use a steam autoclave. Do not exceed 134° C.</li> <li>• Steam autoclaving – Full cycle: wrapped or unwrapped at 134°C, 2 bar/273°F, 29 psi for 3 minutes.</li> </ul>
<b>Drying</b>	<ul style="list-style-type: none"> <li>• Use the drying cycle of the autoclave. Do not exceed 134°C. Allow the dispenser gun and cartridge to fully dry before storage.</li> </ul>
<b>Maintenance</b>	<ul style="list-style-type: none"> <li>• If the dispenser gun is discolored, damaged, worn, or distorted it should be discarded.</li> <li>• No additional maintenance or lubrication is recommended.</li> </ul>
<b>Storage</b>	<ul style="list-style-type: none"> <li>• Store the dispenser gun and cartridge at room temperature, away from moisture or excessive humidity. Dispenser gun and cartridge and attached mixing tip should be disinfected and dried prior to storage.</li> <li>• Insert slide through the front, engaging advancing teeth.</li> </ul>
<b>Manufacturer Contact</b>	<ul style="list-style-type: none"> <li>• Within the United States, call Dentsply Sirona at 1-302-422-4511. For areas outside the United States, contact your local Dentsply Sirona representative.</li> </ul>

### Instructions for disinfecting/sterilizing the digit® Targeted Delivery System syringe dispenser

<b>Warnings</b>	<ul style="list-style-type: none"> <li>• High level disinfection by itself is not appropriate for the digit® syringe.</li> <li>• It is not recommended to submerge digit® syringe into disinfection solutions.</li> <li>• Do not allow the device to exceed 134°C.</li> </ul>
<b>Limitations on Reprocessing</b>	<ul style="list-style-type: none"> <li>• Repeated processing has minimal effect on the digit® syringe.</li> <li>• The digit® syringe can be subjected to the manual cleaning and disinfection process up to 50 times.</li> <li>• The digit® syringe can be subjected to the automated washer/disinfectant process up to 1500 times.</li> <li>• The digit® syringe can be subjected to the steam autoclave sterilization process up to 50 times.</li> <li>• Cold liquid disinfection/sterilization, chemical vapor sterilization, and dry heat sterilization methods have not been tested or validated for efficacy and are not recommended for use.</li> </ul>
<b>Point of Use</b>	<ul style="list-style-type: none"> <li>• Remove excess soil with disposable cloth/paper wipe.</li> <li>• It is recommended that the digit® syringe is reprocessed as soon as is reasonably practical following use.</li> <li>• Prior to cleaning/disinfection/sterilization, the digit® syringe dispenser should be disassembled:               <ol style="list-style-type: none"> <li>1. Remove the plunger by withdrawing fully out the back.</li> <li>2. Remove excess material with a soft paper tissue and alcohol of 70%.</li> </ol> </li> </ul>
<b>Containment and Transportation</b>	<ul style="list-style-type: none"> <li>• No particular requirements.</li> </ul>
<b>Cleaning and Disinfection: Automated</b>	<ul style="list-style-type: none"> <li>• Use only a properly maintained, inspected, calibrated, and approved washer/disinfectant (according to ISO 15883).</li> <li>• Perform a cleaning program cycle (the following cycle is recommended):               <ul style="list-style-type: none"> <li>• 1 minute pre-wash/rinse in cold tap water.</li> <li>• 5 minutes wash with enzymatic detergent at 55°C.</li> <li>• 1 minute enzymatic neutralization in cold tap water.</li> <li>• 1 minute rinse in cold tap water.</li> <li>• Thermal disinfection with minimum AO of 3000 (e.g. 5 minutes at 90°C).</li> <li>• 20 minutes drying at 100°C.</li> </ul> </li> <li>• Follow manufacturer’s recommendation for use of detergent and neutralizer, observing concentrations and contact times.</li> </ul>
<b>Cleaning and Disinfection: Manual</b>	<ul style="list-style-type: none"> <li>• The digit® syringe has to be cleaned by scrubbing with hot water and soap or detergent.</li> <li>• After cleaning thoroughly wipe all device surfaces with a single-use cloth in combination with an alcohol-based, bactericidal, virucidal and fungicidal disinfection solution approved according to local regulations and use according to disinfectant solution manufacturer’s Instructions for Use. Make sure disinfectant solution is compatible with cleaning solution. Pay special attention to device seams and insertions.</li> <li>• Remove disinfectant solution residue with a cloth soaked with water.</li> <li>• Dry device with a lint-free single-use cloth.</li> </ul>
<b>Packaging</b>	<ul style="list-style-type: none"> <li>• Paper/plastic steam sterilization pouches may be used, but are not required.</li> </ul>
<b>Sterilization</b>	<ul style="list-style-type: none"> <li>• Use a steam autoclave. Do not exceed 134°C.</li> <li>• Steam autoclaving – Full cycle: wrapped or unwrapped at 134°C, 2 bar/273°F, 29 psi for 3 minutes.</li> </ul>
<b>Drying</b>	<ul style="list-style-type: none"> <li>• Use the drying cycle of the autoclave. Do not exceed 134°C.</li> <li>• Allow the digit® syringe to fully dry before reassembly and/or storage.</li> <li>• Insert plunger into the syringe cylinder.</li> <li>• Press components together.</li> </ul>

Maintenance	<ul style="list-style-type: none"> <li>• If the digit® syringe is damaged, worn, or distorted it should be discarded.</li> <li>• No additional maintenance or lubrication is recommended.</li> </ul>
Storage	<ul style="list-style-type: none"> <li>• Store the digit® syringe at room temperature, away from moisture or excessive humidity. Syringe should be disinfected/sterilized and dried prior to storage.</li> </ul>
Manufacturer Contact	<ul style="list-style-type: none"> <li>• Within the United States, call Dentsply Sirona at 1-302-422-4511. For areas outside the United States, contact your local Dentsply Sirona representative.</li> </ul>

**Technique Tip:** Unset Aquasil Ultra+ Material can be removed from clothing with dry cleaning solvent.

#### 4.2 Disinfection of the Impression Material

The impression should be disinfected with a hospital-level tuberculocidal disinfectant according to national and local regulations.

- Iodophors, sodium hypochlorite (0.525%), chlorine dioxide, and dual or synergized quaternary ammoniums are approved disinfectants.
- Thoroughly soak by spraying or immerse the impression in any recommended hospital level disinfectant for the contact time recommended by the disinfectant manufacturer for optimum results.
- Water-based disinfectant solutions are preferred.

#### 4.3 Casting

- The impression should be removed from the disinfectant, rinsed with water, and lightly air dried.
- The impression may be poured in plaster or stone 30 minutes after disinfection or delayed up to fourteen days. For epoxy dies, delay pour 60 minutes.
- Do not store impression in direct sunlight.
- If the impression is to be shipped, use suitable packaging to preclude distortion.
- The material is compatible with a range of dental plasters and die stones. Impressions may be silver or copper-plated.

#### 5. LOT NUMBER AND EXPIRATION DATE

1. Do not use after expiration date. ISO standard uses: "YYYY/MM."
2. The following numbers should be quoted in all correspondences:
  - Reorder number
  - Lot number
  - Expiration date

Manufactured by  
Dentsply Caulk  
38 West Clarke Avenue  
Milford, DE 19963 USA  
Tel.: 1-302-422-4511  
www.dentsplysirona.com

**EC REP**  
Dentsply De Trey GmbH  
De-Trey-Str. 1  
78467 Konstanz  
Germany  
Tel.: 49-7531-583-0  
www.dentsplysirona.com

Distributed by  
Dentsply Canada  
161 Vinyl Court  
Woodbridge, Ontario  
L4L 4A3 Canada

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