Ceram·X[®]

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Ceram·X®

Universal Nano-Ceramic Restorative

CAUTION: For dental use only.

USA: Rx only.

Content	Page
1 Product description	2
2 Safety notes	3
3 Step-by-step instructions	5
4 Hygiene	8
5 Lot number and expiration date	9

Note: These Instructions for Use apply to all **Ceram·X®** restorative materials such as **Ceram·X® mono** restorative (including **Ceram·X® mono+** restorative) and **Ceram·X® duo** restorative (including **Ceram·X® duo+** restorative).

1 Product description

Ceram·X® restorative is a light cured, radiopaque restorative material for restoration of anterior and posterior teeth. Ceram·X® restorative combines Nanotechnology with improved organically modified Ceramic particles, resulting in a Nano-Ceramic restorative with unique features: Natural esthetics by simple procedure, extraordinary low monomer release and superior handling characteristics.

Ceram·X® restorative provides two shading systems in one product to cover different clinical and esthetic demands:

Ceram·X® mono restorative, the Single Translucency System, comprises seven shades of intermediate translucency comparable to conventional composites optimal for fast and easy restorations of anterior and posterior teeth.

Ceram·X® duo restorative, the Double Translucency System, offers four dentin shades with translucencies of natural dentin and three enamel shades which mimic natural enamel. Their design has been optimized for highly esthetic restorations with a minimum number of shades. Additionally, Ceram·X® duo restorative comprises one bleach dentin shade for the restoration of bleached teeth.

The **Ceram·X® i-shade label** eases shade selection, covering the whole VITA®¹ classical shade range for both the Ceram·X® mono restorative and Ceram·X® duo restorative system.

1.1 Delivery forms

- Predosed Compules® Tips
- Syringes

¹ Not a registered trademark of Dentsply Sirona, Inc.

1.2 Composition

- Methacrylate modified polysiloxane (organically modified ceramic)
- Dimethacrylate resins
- Fluorescent pigment
- UV stabilizer
- Stabilizer
- Camphorquinone
- Ethyl-4(dimethylamino)benzoate
- Barium-aluminium-borosilicate glass
- Silicon dioxide nano filler
- Iron oxide pigments, titanium oxide pigments and aluminium sulfo silicate pigments according to shade

1.3 Indications

- Direct restorations of all cavity classes in anterior and posterior teeth.

1.4 Contraindications

- Use with patients who have a history of severe allergic reaction to methacrylate resins or any of the other components.

1.5 Compatible adhesives

The material is to be used following application of a compatible (meth)acrylate-based dentin/enamel adhesive such as Dentsply Sirona adhesives designed for use with visible light cured composite restoratives (e.g. the etch and rinse adhesives XP Bond®, Prime&Bond® XP and Prime&Bond® NT or Xeno®, self-etching adhesives).

2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

2.1 Warnings

The material contains methacrylates and polymerizable monomers which may be irritating to skin, eyes and oral mucosa and may cause sensitization by skin contact and allergic contact dermatitis in susceptible persons.

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes, rinse with plenty of water and seek medical attention.
- **Avoid skin contact** to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, remove material with cotton and alcohol and wash thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.
- **Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, remove material from the tissues. Flush mucosa with plenty of water and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.

2.2 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use. Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- If refrigerated, allow material to reach room temperature prior to use.
- Use protective measures for the dental team and patients such as glasses and rubber dam in accordance with local best practice.
- The syringes and Compules® Tips cannot be reprocessed. To prevent syringes (and Compules® Tips when used for indirect application) from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the devices are handled offside the dental unit with clean/disinfected gloves. Discard devices if contaminated.
- Compules® Tips may be used for direct intraoral application of restorative material into a cavity or for indirect application by first placing the restorative material on a pad. After intraoral use, discard Compules® Tips and do not reuse in other patients in order to prevent cross-contamination.
- Use of Compules® Tips with the Compules® Tips Gun is recommended.
- The Compules® Tips Gun is only intended to be used with Dentsply Sirona Compules® or Compula® Tips.
- For further information please refer to the Compules® Tips Gun Instructions for Use.
- The shade guide and the individual shade guide tabs cannot be reprocessed. To prevent the shade guide and the individual shade guide tabs from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the shade guide and the individual shade guide tabs are handled with clean/disinfected gloves.
- The material should extrude easily. DO NOT USE EXCESSIVE FORCE. Excessive pressure may cause the Compules® Tips to rupture or to eject from the Compules® Tips Gun.
- Tightly close syringes immediately after use.
- Contact with saliva, blood and sulcus fluid during application may cause failure of the restoration. Use adequate isolation such as rubber dam.
- Interactions:
 - Do not use eugenol- or hydrogen peroxide-containing materials in conjunction with this product since they may interfere with hardening of the product.

2.3 Adverse reactions

- Eye contact: Irritation and possible corneal damage.
- Skin contact: Irritation or possible allergic response. Reddish rashes may be seen on the skin.
- Contact with mucous membranes: Inflammation (see Warnings).

2.4 Storage conditions

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

- Store in sealed packs at temperatures between 6 °C and 28 °C (43 °F and 82 °F).
- Keep out of direct sunlight and protect from moisture.
- Do not freeze.
- Do not use after expiration date.

Humidity can adversely affect the properties of the material. For unsealed Compules® Tips and unsealed syringes, the following is recommended:

- Store in a dry environment (< 80% relative humidity).
- Use within 3 months.

3 Step-by-step instructions

3.1 Shade selection

Ceram-X® restorative offers two different shade systems:

- Ceram-X® mono restorative offers seven shades of intermediate translucency (M1-M7) and is especially suitable for fast, single-shade treatment procedures.
- Ceram-X® duo restorative offers four dentin shades of low translucency (D1-D4) and three enamel shades of high translucency (E1-E3). With these shades you may copy every VITA® shade. Ceram-X® duo restorative is especially suitable for esthetically demanding situations. Additionally, one bleach dentin shade (DB) is available for bleached teeth.

Decide whether you wish to work with Ceram-X® mono restorative or Ceram-X® duo restorative.

Concerning shade selection procedure there are two possibilities. You can use:

- The VITAPAN®² classical shade guide together with the shade identification label (i-shade label). With the enclosed i-shade label you may select the suitable Ceram-X® mono restorative or duo shade(s) corresponding to the VITA® shade range.
- The Ceram-X® shade tabs which are made from original Ceram-X® restorative material.

3.1.1 Shade selection with VITAPAN® classical shade guide and i-shade label

1. Fix the delivered i-shade label at the rear side of the VITAPAN® classical shade guide.
2. Select the VITAPAN® classical shade to be matched.
3. Select the corresponding Ceram-X® shade according to the recipe on the i-shade label.

Ceram-X® mono restorative

Each VITA® shade is reproduced by one of the seven Ceram-X® restorative mono shades.

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Ceram-X® duo restorative

Each VITA® shade is reproduced by a combination of one dentin shade and one enamel shade.

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² Not a registered trademark of Dentsply Sirona, Inc.

3.1.2 Shade selection with delivered Ceram-X® shade tabs

To select the tooth color you may use the delivered shade guides: one of each is available for Ceram-X® mono restorative and Ceram-X® duo restorative.

Avoid contamination of the shade guide.

To prevent the shade guide from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the shade guide is handled offside the dental unit with clean/disinfected gloves. For shade selection, remove individual tabs from shade guide holder.

Ceram-X® mono restorative

1. Select the Ceram-X® shade most closely matching the color of the tooth. Consider the central region of the tooth.

Ceram-X® duo restorative

1. Select the dentin shade most closely matching the color of the moist dentin visible in the cavity or consider the cervical region of the tooth.
2. Select the enamel shade most closely matching the color of the tooth. Consider the incisal, occlusal or proximal region of the tooth.

3.1.3 Shade selection hints

1. Clean the tooth surface with prophylaxis paste to remove plaque or surface stain.
2. Select the shade while teeth are hydrated.
3. When working with Ceram-X® duo restorative you should select a slightly darker rather than a lighter dentin shade. The succeeding enamel layer tends to lighten the optical result.

3.2 Cavity preparation

1. Clean the tooth surface with a fluoride free prophylaxis paste (e.g. Nupro® prophylaxis paste).
2. Prepare cavity (unless not necessary, e.g. cervical lesion).
3. Use adequate isolation such as rubber dam.
4. Rinse surface with water spray and carefully remove rinsing water. Do not desiccate the tooth structure.

3.2.1 Matrix placement and wedging (Class II restorations)

1. Place a matrix (e.g. AutoMatrix® or Palodent® matrix system) and wedge. Burnishing of the matrix band will improve contact and contour. Pre-wedging or BiTine® ring placement is recommended.

3.3 Pulp protection, tooth conditioning/dentin pre-treatment, adhesive application

Refer to adhesive manufacturer's Instructions for Use for pulp protection, tooth conditioning and/or adhesive application. Once the surfaces have been properly treated, they must be kept uncontaminated. Proceed immediately to placement of the material.

3.4 Application

Apply material in increments immediately after the application of the adhesive. The material is designed to be cured in increments up to a 2 mm depth/thickness.

Using Compules® Tips

1. Insert Compules® Tip into the notched opening of the Compules® Tips Gun. Be certain that the collar on the Compules® Tip is inserted first.
2. Remove the colored cap from the Compules® Tip. The Compules® Tip may be rotated 360° to gain the proper angle of entrance into the cavity or to the mixing pad.

DO NOT USE EXCESSIVE FORCE.

Indirect use: to prevent Compules® Tip from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the Compules® Tip is handled offside the dental unit with clean/disinfected gloves. Dispense the necessary amount of material onto a mixing pad and protect against light. Apply material into the cavity with a plastic instrument.

- or -

Direct intraoral use: dispense the material directly into the cavity preparation using a slow, steady pressure.

3. To remove the used Compules® Tip, be sure that the Compules® Tips Gun plunger is pulled back completely by allowing the handle to open to its widest position. Apply a downward motion to the front end of the Compules® Tip and remove.

Excessive force.

Injury.

1. Apply slow and steady pressure on the Compules® Tips Gun.
2. Do not use excessive force. Compules® Tip rupture or ejection from Compules® Tips Gun may result.

Using syringes

Avoid contamination of the syringe.

To prevent the syringe from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringe is handled offside the dental unit with clean/disinfected gloves.

1. Remove the cap.
2. Turn the handle of the syringe slowly in a clockwise direction and dispense the necessary amount of the material onto a mixing pad.
3. Point the front Tip of the syringe upwards and turn the handle anti-clockwise to prevent oozing of the material.
4. Immediately close the syringe with the cap.
5. Protect the restorative material on the mixing pad against light.

3.4.1 Applying Ceram-X® mono restorative

1. Fill the whole cavity incrementally. Each layer ≤ 2 mm.
2. Cure each layer according to the table below [3.5 Light curing].

3.4.2 Applying Ceram-X® duo restorative

Rebuild the tooth with respect to its anatomical structure. Consider that depending on the thickness of the enamel and dentin layers, the optical result may vary from the selected VITA® shade.

1. Rebuild the dentin with the selected dentin shade (thickness of increment ≤ 2 mm).
2. Cure each increment of dentin according to the table below [3.5 Light curing].
3. Rebuild the enamel layer with the selected enamel shade (thickness of increment ≤ 2 mm).
4. Cure each increment of enamel according to the table below [3.5 Light curing].

3.5 Light curing

1. Light cure each area of the restoration surface with a suitable curing light³. Minimum light output must be at least 500 mW/cm², exposure time according to the shade as listed in the table below. Refer to curing light manufacturer's Instructions for Use for compatibility and curing recommendations.

Insufficient curing.

Inadequate polymerization.

1. Check compatibility of curing light.
2. Check curing cycle.
3. Check minimum irradiance.
4. Cure each area of each increment for the recommended curing time.

Curing time for 2 mm increments

Output	≥ 500 mW/cm ²	≥ 800 mW/cm ²
Ceram-X [®] mono shades (M1-M7)	20 sec	20 sec
Ceram-X [®] duo dentin shades (D1-D4 and DB)	40 sec	30 sec
Ceram-X [®] duo enamel shades (E1-E3)	10 sec	10 sec

3.6 Finishing and polishing

1. Contour the restoration using finishing burs or diamonds.
2. Use Enhance[®] finishing devices for additional finishing.
3. Prior to polishing, check that surface is smooth and free of defects.
4. For achieving a very high luster of the restoration, we recommend ceram.x[®] gloss, Enhance[®] and PoGo[®], finishing and polishing instruments, Prisma[®] Gloss™ Regular and Extrafine polishing paste.

For contouring, finishing, and/or polishing, follow the manufacturer's Instructions for Use.

4 Hygiene

4.1 Compules[®] Tips Gun

For reprocessing instructions please refer to the Instructions for Use of the Compules[®] Tips Gun, which is available on our webpage at www.dentsply.eu/IFU. If requested, we will send you a free printed copy of the Instructions for Use in the language you require within 7 days. You can use the order form provided on our webpage for this purpose.

³ Curing light designed to cure materials containing camphorquinone (CQ) initiator. Peak of spectrum in the range of 440-480 nm.

4.2 Compules® Tips – cross-contamination

Cross-contamination.

Infection.

1. Do not reuse Compules® Tips after intraoral use and/or if contaminated.
2. Dispose the intraorally used and/or contaminated Compules® Tips in accordance with local regulations.

4.3 Syringes – cross-contamination

Cross-contamination.

Infection.

1. The syringes cannot be reprocessed.
2. To prevent the syringes from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringes are handled with clean/disinfected gloves. Do not reuse syringes if contaminated.
3. Dispose contaminated syringes in accordance with local regulations.

4.4 Shade guide and individual shade guide tabs – cross-contamination

Cross-contamination.

Infection.

1. The shade guide and the individual shade guide tabs cannot be reprocessed.
2. To prevent the shade guide and the individual shade guide tabs from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the shade guide and the individual shade guide tabs are handled with clean/disinfected gloves. Do not reuse shade guide and the individual shade guide tabs if contaminated.
3. Dispose contaminated shade guide and the individual shade guide tabs in accordance with local regulations.

5 Lot number () and expiration date ()

1. Do not use after expiration date.
ISO standard is used: “YYYY-MM” or “YYYY-MM-DD”.
2. The following numbers should be quoted in all correspondence:
 - Reorder number
 - Lot number
 - Expiration date

© Dentsply Sirona 2016-04-15

[These Instructions for Use are based on Master Version 12]

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Dentsply DeTrey GmbH