For better dentistry

**DENTSPLY**

# AH Plus® Cleaner

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# AH Plus® Cleaner

## Liquid For Removing Unset AH Plus® Root Canal Sealer

Caution: For dental use only.

In the USA: Rx only

Content	Page
1 Product description.....	2
2 Safety notes .....	3
3 Step-by-step instructions .....	4
4 Hygiene.....	5
5 Lot number and expiration date.....	5

### 1 Product description

**AH Plus® Cleaner** liquid is used when root canal filling and adhesive restoration of a tooth are performed during one appointment.

**AH Plus® Cleaner** liquid is designed for the removal of residual unset AH Plus® root canal sealer from the cavity walls.

**AH Plus® Cleaner** liquid offers the following features:

- Removes residuals of unset AH Plus® root canal sealer from enamel and dentin.
- Prepares enamel and dentin for application of XP BOND® Universal Total-Etch Adhesive, Prime&Bond® NT Nano-Technology Dental Adhesive and Xeno® V+ One Component Self-Etching Dental Adhesive.

#### 1.1 Delivery forms

- Easy squeeze bottle

#### 1.2 Composition

- Ethanol
- Tertiary butanol
- Water

#### 1.3 Indications

- Removal of residuals of unset AH Plus® root canal sealer from the cavity walls prior to adhesive restorative treatment.

#### 1.4 Contraindications

- None known.

## 2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

### Safety alert symbol

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

### 2.1 Warnings

The material contains tertiary butanol which may be irritating to skin and eyes.

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes rinse immediately with plenty of water and seek medical attention.
- **Avoid skin contact** to prevent irritation. If contact with skin occurs, immediately wash with soap and water and rinse thoroughly.

### 2.2 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use.

Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- Use protective measures for the dental team and patients such as glasses and rubber dam in accordance with local best practice.
- Contact with saliva, blood and sulcus fluid during application may cause failure of the restoration. Use adequate isolation such as rubber dam.
- Tightly close bottles immediately after use.
- Flammable: Keep away from sources of ignition. Take precautionary measures against static discharges.
- Foam pellets are intended for single use only. Discard after use. Do not reuse in other patients in order to prevent cross-contamination.

### 2.3 Adverse reactions

- Eye contact: Irritation and possible corneal damage.
- Skin contact: May cause skin irritation.

### 2.4 Storage conditions

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

- Store the product at temperatures between 2 °C and 28 °C (36 °F and 82 °F). Use the product at room temperature.
- Do not freeze.
- Do not use after expiration date.

### 3 Step-by-step instructions

#### 3.1 Preparation

1. Perform root canal filling with gutta-percha points and AH Plus® root canal sealer in such a way that the coronal part of the root canal is mainly obturated by gutta-percha (e.g. by use of additional gutta-percha points).
2. Remove excess length of the points with hot instrument (excavator).
3. Immediately condense the still warm gutta-percha with a hand instrument (ball-burnisher) in an apical direction. AH Plus® root canal sealer is pressed out of the root canal and the coronal end of the root canal filling now completely consists of gutta-percha.

#### 3.2 Dosage

##### Easy squeeze bottle

1. Dispense ample amounts of AH Plus® Cleaner liquid into a clean Applicator Dish mixing well or standard dappen dish or mixing well. If necessary, evaporation may be avoided by covering one Applicator Dish mixing well by another.
2. Replace cap promptly.

#### 3.3 Application

1. Remove all of residual of unset AH Plus® root canal sealer from floor and walls of the access cavity with foam pellets soaked with AH Plus® Cleaner liquid.
2. Repeat procedure at least two times with unused foam pellets soaked with AH Plus® Cleaner liquid, until no more AH Plus® root canal sealer residue is visible.
3. Remove remains of AH Plus® Cleaner liquid with vigorous water spray and rinse cleaned areas thoroughly for at least 5 seconds.
4. Remove rinsing water completely by blowing gently with an air syringe.

##### Incomplete removal of:

1. Unset AH Plus® root canal sealer
2. AH Plus® Cleaner liquid

Reduced adhesion.

1. Remove unset AH Plus® root canal sealer as completely as possible from enamel and dentin.
2. Remove remains of AH Plus® Cleaner liquid as completely as possible from enamel and dentin.

#### 3.3.1 Direct adhesive restoration

1. Immediately continue with the restorative procedure.
2. Apply Xeno® V+ Self-Etching Adhesive or DeTREY® Conditioner 36 etching gel followed by Prime&Bond® NT or XP BOND® etch and rinse adhesives. Refer to adhesive manufacturer's Instructions for Use for adhesive application.
3. Immediately continue with placement of the restorative material. The use of SDR™ Smart Dentin Replacement restorative is recommended. It is designed to be used as a base in Class I and II restorations. Proceed immediately to complete the restoration using a methacrylate based universal/posterior restorative material. Refer to the manufacturer's Instructions for Use for application.

## 4 Hygiene

### Cross-contamination

Infection.

1. Do not reuse single use products. Dispose in accordance with local regulations.
2. Reprocess reusable products as described below.

### 4.1 Applicator Dish mixing well

#### 4.1.1 Cleaning

1. Remove residual AH Plus® Cleaner liquid with tissue.
2. Clean by scrubbing with water and soap.

#### 4.1.2 Disinfection

1. Disinfect with a water-based hospital-level disinfection solution according to national/local regulations.
2. Follow the disinfectant manufacturer's Instructions for Use.
3. Rinse and dry.

#### 4.1.3 Sterilization (optional)

1. Items can be sterilized by steam autoclave at 134 °C/2 bar = 273 °F/29 psi, unwrapped; sterilization time 3 minutes 30 seconds.

Items can be disinfected for up to 50 times and can be sterilized for up to 8 times.

## 5 Lot Number ( ) and Expiration Date ( )

1. Do not use after expiration date. ISO standard is used: "YYYY/MM"
2. The following numbers should be quoted in all correspondence:
  - Reorder Number
  - Lot number
  - Expiration date

© DENTSPLY DeTREY 2011-03-14

[These Instructions for Use are based on Master Version 04]

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**DENTSPLY Limited**
**DENTSPLY Italia S.r.l.**
**DeguDent Austria GmbH**
**DENTSPLY DeTrey GmbH**