Fix Adhesive

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Fix Adhesive

Tray Adhesive for Alginate Impression Materials

Fix Solvent

Cleanser for Removal of Fix Adhesives

CAUTION: For dental use only.

USA: Rx only.

Content	Page
1 Product description	2
2 Safety notes	3
3 Step-by-step instructions	4
4 Lot number and expiration date	4

1 Product description

Fix Adhesive is a solvent-based adhesive which increases the adhesion between alginate impression materials and the impression tray surface and is available as liquid or as spray.

Fix Solvent is a spray to remove any remaining adhesive from the impression tray after impression taking and casting.

1.1 Delivery forms

Fix Adhesive – Liquid

- Bottle

Fix Adhesive – Spray

- Spray

Fix Solvent

- Spray

1.2 Composition

Fix Adhesive – Liquid

- Diethylenetriamine polymer
- Propan-2-ol
- Xylene
- Erythrosine

Fix Adhesive – Spray

- Diethylenetriamine polymer
- Propan-2-ol
- Xylene
- Erythrosine
- Propellant: Butane

Fix Solvent

- Propan-2-ol
- Xylene
- Propellant: Butane

1.3 Indications

Fix Adhesive

- Tray adhesives for alginate impression materials.

Fix Solvent

- Removal of Fix Adhesives from the impression tray after impression taking and casting.

1.4 Contraindications

- Use in patients with a known allergy to diethylenetriamine or any of the other components.

2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

2.1 Warnings

The material contains diethylenetriamine which may be irritating to skin, eyes and oral mucosa and may cause sensitization allergic contact dermatitis in susceptible persons.

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes, rinse with plenty of water and seek medical attention.
- **Avoid skin contact** to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, remove material with cotton and alcohol and wash thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.
- **Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, remove material from the tissues. Flush mucosa with plenty of water and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.
- **Do not breathe vapor or spray.**

2.2 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use. Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- Use protective measures for the dental team and patients such as glasses in accordance with local best practice.
- Tightly close bottle immediately after use.
- Use in a well-ventilated area. Avoid inhaling vapor.
- To prevent the bottle from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the bottle is handled offside the dental unit with clean/disinfected gloves.
- Apply adhesive only to uncontaminated impression trays.
- Fix Adhesive – Liquid is highly flammable, Fix Adhesive – Spray and Fix Solvent are extremely flammable: Keep away from sources of ignition.
- Interactions:
 - Fix Adhesive forms an adhesive bond between unset alginate impression material and the surface of the impression tray.

2.3 Adverse reactions

- Eye contact: Irritation and possible corneal damage.
- Skin contact: Sensitization. Irritation or possible allergic response. Reddish rashes may be seen on the skin.
- Contact with mucous membranes: Inflammation (see Warnings).

2.4 Storage conditions

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

- Keep the product out of direct sunlight and store in a well-ventilated place.
- Store at temperatures between 10 °C and 24 °C (50 °F and 75 °F). Use the product at room temperature.
- Keep container tightly closed.
- Do not pierce or burn even after use.
- Do not use after expiration date.

3 Step-by-step instructions

Clogging of spray nozzle.

Before using the spray, make sure that the spray nozzle is not clogged. Use protective measures for the dental team such as glasses in accordance with local best practice.

3.1 Application

1. Select a suitable impression tray.
2. Prior to mixing the alginate impression material, apply a thin coat of Fix Adhesive onto the inner surface of the tray. Do not extend over the border of the tray onto surfaces which will not be covered by the impression material.
3. Allow the adhesive layer to dry for at least 3 minutes to become tacky.
4. Load the tray with alginate impression material in the usual way.

3.2 Cleaning

1. After impression taking and casting remove alginate impression material from impression tray.
2. To remove any remaining adhesive, spray Fix Solvent on the inner surface of the impression tray. Leave for some seconds to dissolve the remaining adhesive.
3. Wipe off residual solvent with a paper towel.

4 Lot number () and expiration date ()

1. Do not use after expiration date.
ISO standard is used: "YYYY-MM" or "YYYY-MM-DD".
2. The following numbers should be quoted in all correspondence:
 - Reorder number
 - Lot number
 - Expiration date

© Dentsply Sirona 2014-01-31

[These Instructions for Use are based on Master Version 01]

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Dentsply DeTrey GmbH