Poly-F[®] Plus

Zinc Polycarboxylate Cement

Zinkpolycarboxylatzement

Ciment polycarboxylate de zinc

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CAUTION: For dental use only.

USA: Rx only

Content	Page
1 Product description	2
2 Safety notes	3
3 Step-by-step instructions	4
4 Lot number and expiration date	5

1 Product description

Poly-F[®] Plus is a conventional zinc polycarboxylate cement.

Poly-F[®] Plus is presented as a powder containing all reactive components. This powder, which is available in one distinctive shade, is mixed with demineralized water only.

Poly-F[®] Plus cement complies with the requirements of ISO 9917 for zinc polycarboxylate cements.

1.1 Delivery forms

A powder/liquid version for hand mixing.

1.2 Composition

- Zinc oxide
- Magnesium hydroxide
- Calcium-alumino-sodium-fluoro-phosphor-silicate glass
- Polyacrylic acid
- Tin fluoride
- Iron oxide pigments

1.3 Indications

- Cementations of crowns, bridges, inlays, and orthodontic bands.
- Cavity lining under all restorative materials.
- Temporary fillings and fillings in deciduous teeth.
- Restorative base.

1.4 Contraindications

- Direct or indirect application to dental pulp (direct or indirect pulp-capping).

2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

2.1 Warnings

The material contains polyacrylic acid and may be irritating to skin, eyes and oral mucosa.

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes rinse with plenty of water and seek medical attention.
- **Avoid skin contact** to prevent irritation. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, remove material with cotton and wash thoroughly with soap and water.
- **Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, remove material from the tissues. Flush mucosa with plenty of water and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.

2.2 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use.

Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- Use protective measures for the dental team and patients such as glasses and rubber dam in accordance with local best practice.
- To prevent bottles, powder scoop and mixing pad from exposure to spatter or spray of body fluids or contaminated hands it is recommended that the bottles, powder scoop and mixing pad are handled offside the dental unit with clean/disinfected gloves.

NOTICE

Moisture uptake.

Moisture can adversely affect the properties of the powder.

1. The desiccant crystals integrated in the cap and/or drying capsule protect the powder from absorbing moisture. The color of the desiccant crystals should be orange. The desiccant crystals are no longer effective after the color has turned from orange to colorless.
2. Tightly close bottle immediately after use.

2.3 Adverse reactions

- Eye contact: Irritation and possible corneal damage.
- Skin contact: Irritation. Reddish rashes may be seen on the skin.
- Contact with mucous membranes: Inflammation (see Warnings).

2.4 Storage conditions

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

- Store with original caps tightly closed in a well-ventilated place at temperatures between 2 °C and 28 °C (35 °F and 82 °F).
- Store in a dry environment (< 80% relative humidity).
- Protect from moisture.
- Do not use after expiration date.

3 Step-by-step instructions

3.1 Pulp protection

1. In deep cavities cover the dentin close to the pulp (less than 1 mm) with a hard-setting calcium hydroxide liner (Dycal® calcium hydroxide liner, see Instructions for Use).

3.2 Dosage and mixing

NOTICE

Premature setting of material.

Reduced working time.

1. Keep powder and liquid separated until ready for mixing.

	Cementation and lining	Base and temporary filling
Mixing ratio	1 scoop powder : 2 drops liquid ¹	2 scoops powder : 2 drops liquid ²
Mixing time	15 seconds	20 to 25 seconds
Working time (from start of mixing)	At least 1 minute 30 seconds	At least 1 minute 30 seconds
Setting time (after end of mixing)	2 minutes 30 seconds to 8 minutes	2 minutes to 6 minutes

1. Fill water dispenser with distilled or de-ionized water.
2. Invert powder bottle to fluff the powder.
3. Overfill the scoop with powder and strike off the excess with the bottle insert leaving a flat surface level with the edge of the scoop. Take care not to press the powder into scoop, as this can give too thick a mixture.
4. Place the powder onto the pad provided or on a glass slab ³.
5. Divide the powder into two halves using a stainless steel spatula.
6. Dispense two drops of water by holding the bottle vertically and squeezing gently.

¹ The same powder : liquid ratio (5 g : 1 g) is proposed for testing purposes according to ISO 9917.

² The same powder : liquid ratio (10 g : 1 g) is proposed for testing purposes according to ISO 9917.

³ Mixing and application of the material should be done at normal room temperature. Higher temperatures shorten the working and setting times of the cement. Under warm conditions or if a long working time is required, it is therefore recommended that the glass slab be cooled. Testing to ISO 9917 is carried out at 23 °C ± 1 °C and a relative humidity of 50 ± 10%.

7. Add one half of the powder to the water. As soon as the first half is wetted, incorporate the second, then mix thoroughly. Mix rapidly to a creamy consistency for cementation and to a putty-like consistency for linings and temporary fillings. For cementation consistency mixing time is 15 seconds. Base consistency may take 5 to 10 seconds longer.

3.3 Finishing

1. Excess material may be removed at once or after completion of setting time (see 3.2 Dosage and mixing). Do not touch the cement when it is in the intermediate, rubbery condition. When used as a temporary restoration, finishing burs are recommended. For removal of surplus material after crown cementation, appropriate hand instruments, like a dental probe, and dental floss are used.

3.4 Cleaning instruments

Zinc polycarboxylate cements adhere to instruments and glass slabs.

1. Immediately after use, either clean instruments with water or drop in disinfectant or water to facilitate subsequent cleaning.

4 Lot number () and expiration date ()

1. Do not use after expiration date. ISO standard is used: “YYYY-MM”
2. The following numbers should be quoted in all correspondence:
 - Reorder Number
 - Lot number
 - Expiration date

© Dentsply Sirona 2015-01-26

[These Instructions for Use are based on Master Version 03]

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Dentsply DeTrey GmbH