Comfortwand™ Universal Sensor Holder

Step-By-Step Instructions

1. Insert corner of sensor into device as indicated (A). Cover edges of sensor with elastic strap to secure sensor (B).
2. Position holder in patient's mouth.
3. Have patient or parent hold device in place.
4. Align x-ray tube.
5. Take exposure.
6. Adjust as needed for the next desired position.

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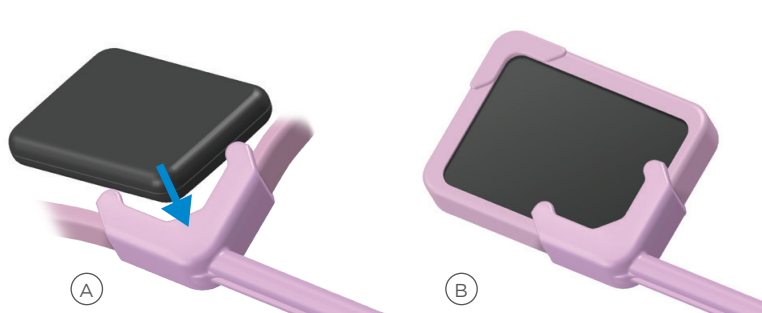
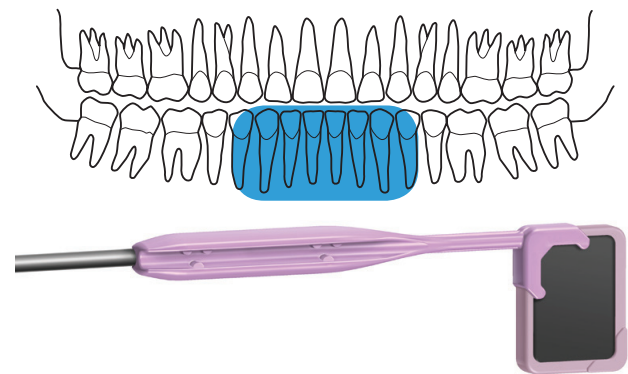
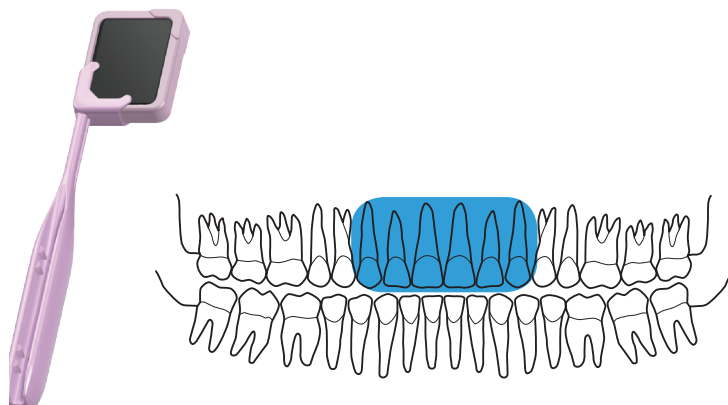
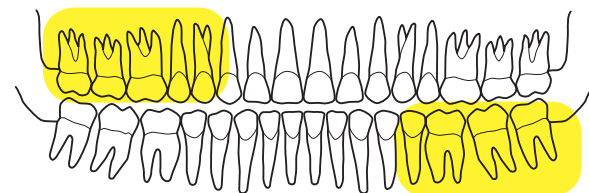
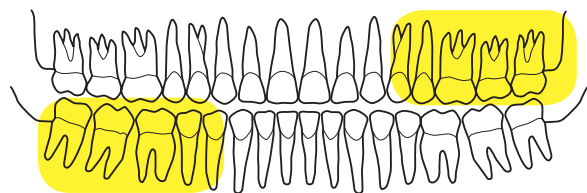
ENGLISH

Caution: U.S. Federal laws restrict this device to sale to or on the order of a dentist.

Indications for use: ComfortWand™ holders are indicated for holding and positioning digital sensors for intraoral dental radiographs.

Contraindications: None known.

Warnings: Sterilize before use. DO NOT use phenol-based glutaraldehyde or ultrasonic cleaners. DO NOT chemically, dry-heat sterilize or cold sterilize. Only sterilize with steam autoclaving according to cleaning and sterilization instructions. It is the user's responsibility to qualify alternative methods of sterilization or disinfection. Failure to follow properly validated sterilization processes and approved aseptic techniques may result in cross contamination.

Precautions: When using with corded digital sensors, use a sensor cover and do not pull on cord during use.

Adverse Reactions: None known.

Reprocessing: Disassemble instrument(s) and separate all parts prior to reprocessing.

Manual Cleaning and Disinfection:

1. Rinse components under running tap water using a soft-bristled brush to remove gross soil using an enzymatic detergent per manufacturer's recommendation, paying special attention to any cavities or holes.
2. Rinse under running tap water and dry with cloth or disposable wipe.
3. After thoroughly cleaning, wipe all surfaces with a disposable wipe in combination with a bactericidal, virucidal, and fungicidal instrument disinfection solution (such as VoloWipe) following manufacturer's instructions. Pay special attention to any cavities or holes.
4. Rinse components by wiping them with a cloth soaked in pure water or by running under running tap water.

Automated Cleaning and Disinfection: As an alternative to manual cleaning and disinfection, the devices can also be processed in a washer-disinfector. Use only properly maintained, calibrated, and approved washer-disinfectors according to ISO 15883-1.

1. Place devices in washer-disinfector allowing water and detergent to wet all surfaces and to drain off afterwards during washer-disinfector program following manufacturer's instructions. Run a washer disinfector program with A0 value ≥ 600 .

Sterilization:

1. Put components in sterilization pouches, and place in the middle tray of the autoclave, away from autoclave walls and heating element. Plastic parts must be in a separate pouch from metal arms to avoid melting or warping.
2. Cycle steam autoclave at 270°F (132°C) for 10 minutes at 30 PSI (206.8 kPa). Do not exceed 273°F (134°C).
3. To maintain sterility, instruments should remain bagged until ready for use.

Storage: Store at room temperature (18-25°C / 65-77°F).

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REF

| | |
|--------|--|
| 550521 | Comfortwand™ (1-pack) |
| 550523 | Comfortwand™ (3-pack) |
| 550500 | Rinn® Universal Relaxed Fit Digital Sensor Covers (500-pack) |
| 550200 | Rinn® Large Digital Sensor Covers (200-pack) |

Use with:

Rinn® Large Sensor Covers 550200

Rinn® Universal Sensor Covers 550500

Dentsply Sirona

Manufactured by Dentsply RINN.
1301 Smile Way
York, PA 17404 U.S.A.
www.dentsplysirona.com
Customer Service: 1-800-989-8825
Rinn®, XCP-DS® and XCP-ORA® are trademarks of
Dentsply Sirona, Inc.
©2017 Dentsply Sirona Preventive. Printed in the USA. 055937 Rev. 3 (0517)

EC REF DENTSPLY Limited
Building 3, The Heights,
Weybridge, Surrey KT13 ONY UK

Dentsply Sirona

Comfortwand™
Universal Sensor Holder