CliXdish™

Mixing Well

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Godet de mélange

Vaschetta miscelatrice

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CAUTION: For dental use only.

USA: Rx only.

Content	Page
1 Product description	2
2 Safety notes	2
3 Step-by-step instructions	3
4 Hygiene	3
5 Lot number and durability	4

1 Product description

CliXdish™ mixing well is a reusable mixing well that features a unique, ambient-light-blocking lid. It protects adhesives and other liquids from ambient light and reduces the evaporation of solvents.

The closed lid serves as a rest for in-use storage of an applicator tip or a similar application device.

2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

2.1 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use.

Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- The CliXdish™ is compatible with alcohol and alcohol-water based Dentsply Sirona adhesives. (For in-use storage time please refer to respective Dentsply Sirona adhesive Instructions for Use.)
- Use protective measures for the dental team and patients such as glasses and rubber dam in accordance with local best practice.
- Please also refer to the Instructions for Use of the products used with this product.

3 Step-by-step instructions

3.1 Assembly and preparation

1. Assemble CliXdish™ by pressing the hinge spindle of the lid into the hinge notch of the mixing well.
2. Touch wing-like part of lid to open or close CliXdish™.

For further information please refer to adhesive Instruction for Use.

4 Hygiene

NOTICE

Wrong cleaning or disinfection method.

Damage to the device.

1. Always disassemble CliXdish™ before reprocessing.
2. Do not clean or disinfect with aggressive agents (e.g. solutions based on orange oil or acetone).
3. Start reprocessing within 1 hour after use.

4.1 Disassembly and preparation for reprocessing

1. Disassemble CliXdish™.
2. Clean CliXdish™ using a soft brush and running cold tap water.
3. Remove residual adhesive with tissue.

4.2 Manual cleaning and disinfection

1. Discard used gloves according to local regulations.
2. Disinfect hands with an appropriate bactericidal, virucidal, and fungicidal hand disinfectant solution according to local regulations. Use according to disinfectant solution manufacturer's Instructions for Use.
3. Use a new clean pair of examination gloves.
4. Clean CliXdish™ with a soft brush under running tap water, and an appropriate cleaning solution (e.g.: alkaline solution: Dürr FD370 [2%]¹) containing nonionic surfactants (5-15%), chelating agents (< 5%) and adjuvants in an aqueous solution, for at least 30 seconds until free of visible contamination. Pay special attention to device seams and insertions.
5. Remove cleaning solution residue with a cloth soaked with deionized water.
6. Dry CliXdish™ with a lint-free single-use cloth.
7. Thoroughly wipe all CliXdish™ surfaces with a single-use cloth in combination with an undiluted, aldehyde-free, alcohol-based, bactericidal, tuberculocidal, virucidal and fungicidal disinfection solution (e.g.: Dürr FD333¹) approved according to local regulations and use according to disinfectant solution manufacturer's Instructions for Use. Make sure disinfectant solution is compatible with cleaning solution. Pay special attention to device seams and insertions.
8. After 5 minutes remove disinfectant solution residue with a cloth soaked with deionized water.
9. Dry CliXdish™ with a lint-free single-use cloth.

Device can be manually disinfected for up to 200 times.

¹ Not a registered trademark of Dentsply Sirona, Inc.

4.3 Automatic Washer-Disinfector

As an alternative to manual cleaning and disinfection, the CliXdish™ can also be processed in a washer-disinfector². Use only properly maintained, calibrated, and approved washer-disinfector according to ISO 15883-1.

1. Place CliXdish™ in the washer-disinfector allowing water and detergent to enter into and drain off from device openings during washer-disinfection program and follow washer-disinfector's Instructions for Use.
2. Run washer-disinfection program with A0 value ≥ 3000 (e.g. 5 min at ≥ 90 °C) using appropriate detergents (e.g.: neodisher® MediClean [0.5%] (alkaline detergent) and neodisher® Z [0.1%] (acid neutralization and cleaning detergent); both detergents from Dr. Weigert, Hamburg, Germany³).

The CliXdish™ can be washer-disinfected for up to 500 times.

4.4 Autoclaving (optional)

After cleaning and disinfection [4.2 Manual cleaning and disinfection] or after processing in a washer-disinfector [4.3 Automatic washer-disinfector], the CliXdish™ can also be autoclaved.

1. Steam autoclave CliXdish™ unwrapped at 134 °C/2 bar = 273 °F/29 psi for 3 minutes 30 seconds.

The CliXdish™ can be disinfected for up to 200 times and can be autoclaved for up to 20 times.

5 Lot number (LOT) and durability

1. Replace CliXdish™ from time to time, depending on the number of reprocessing cycles as mentioned in chapters 4.2 to 4.4.
2. The following numbers should be quoted in all correspondence:
 - Reorder number
 - Lot number
 - Expiration date

© Dentsply Sirona 2016-11-10

[These Instructions for Use are based on Master Version 02]

² For example Miele Washer-Disinfector G7835 CD using program Vario TD.

³ Not registered trademarks of Dentsply Sirona, Inc.

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Dentsply DeTrey GmbH